
TOPRO

strack

Instructions for Use TOPRO Comfort Basic & TOPRO Comfort Plus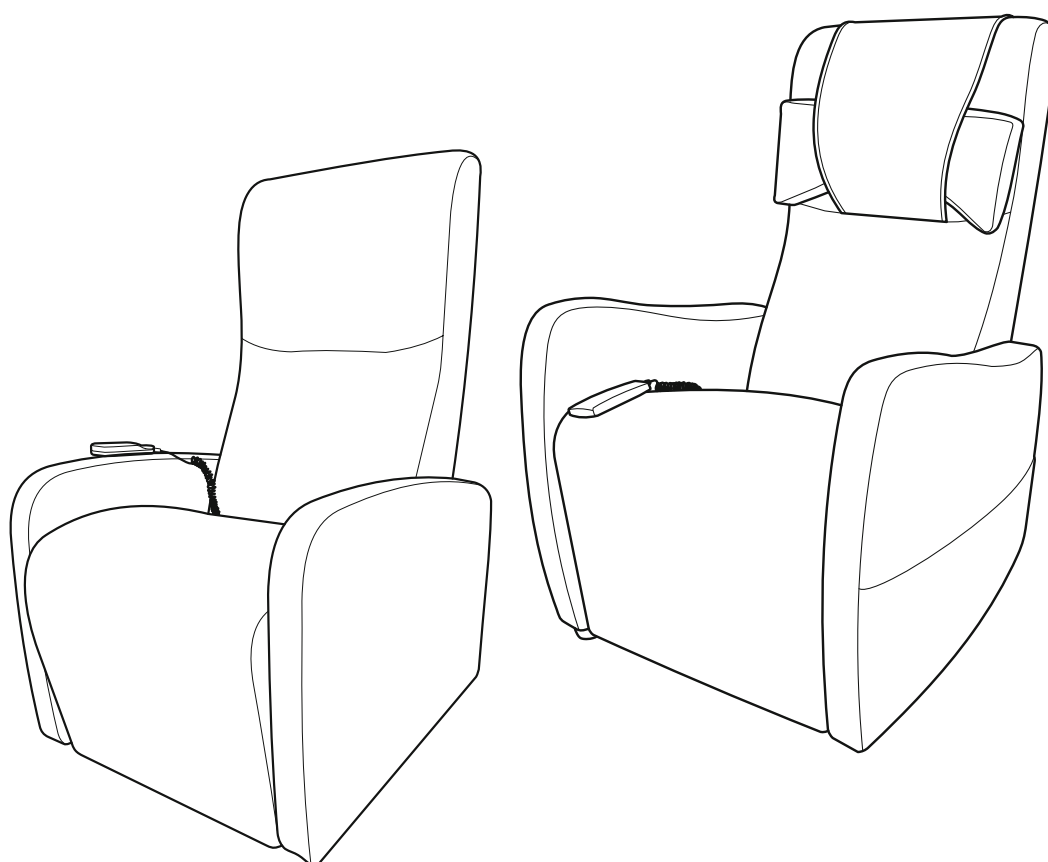

topromobility.com

EN

Illustrations
Dimensions
Technical data
Article numbers

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







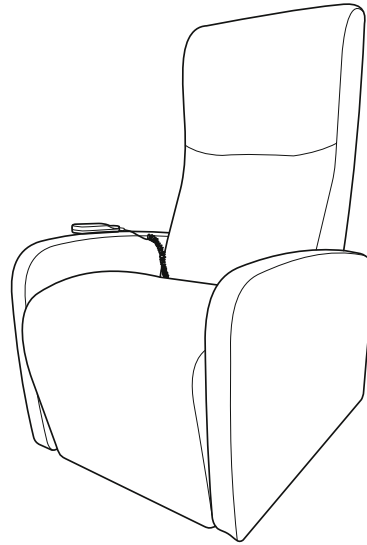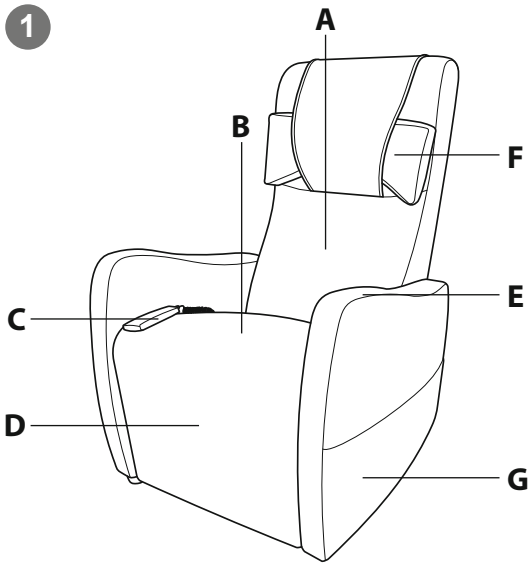


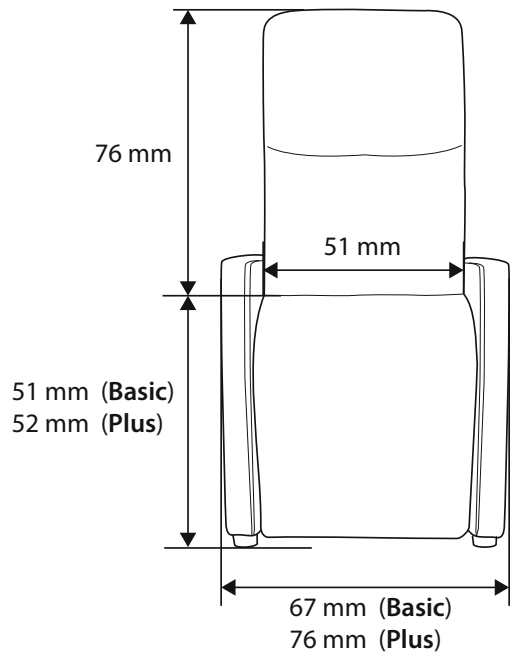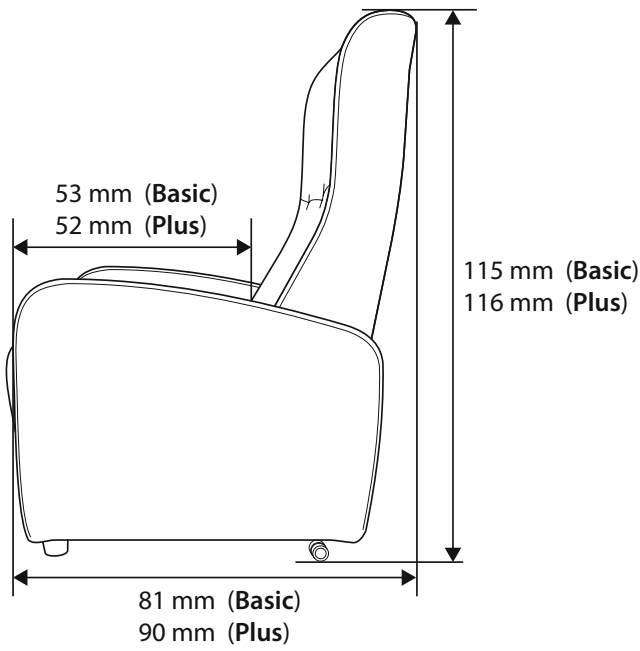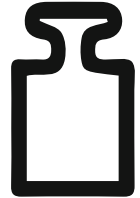
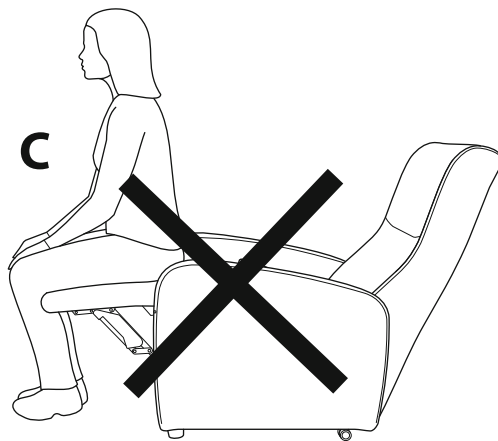
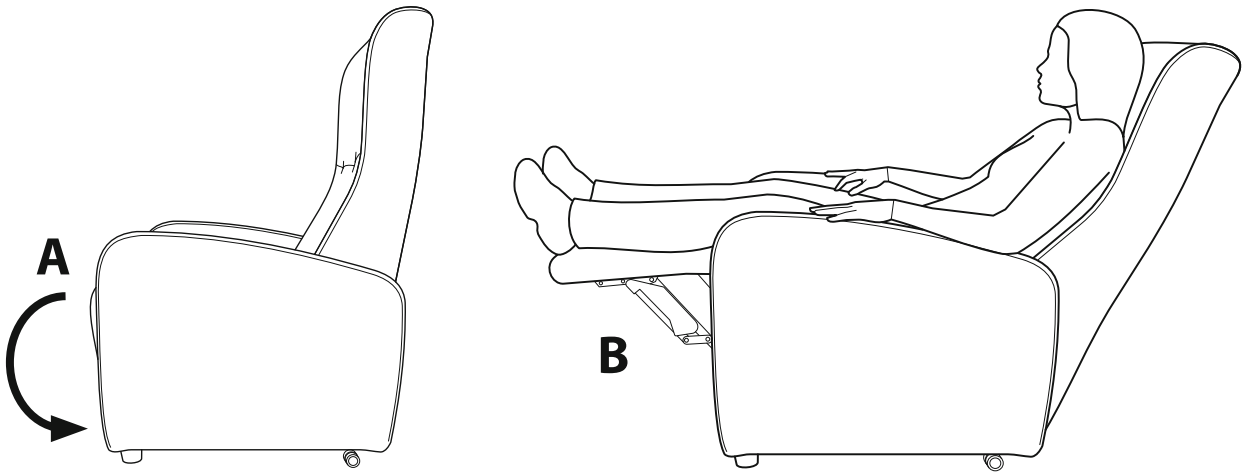
D

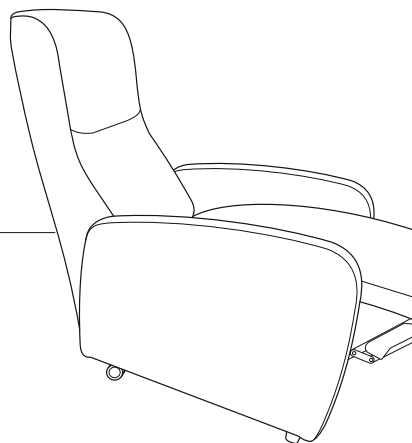E

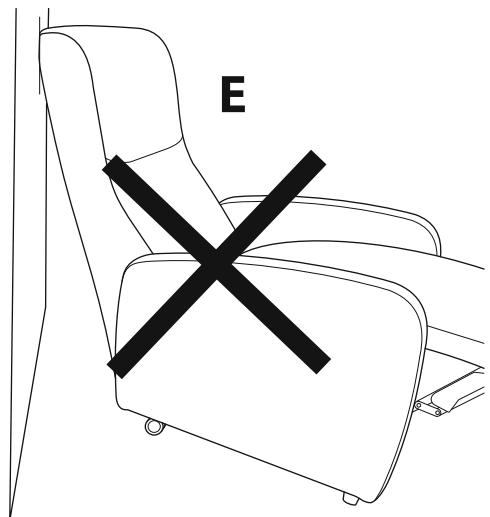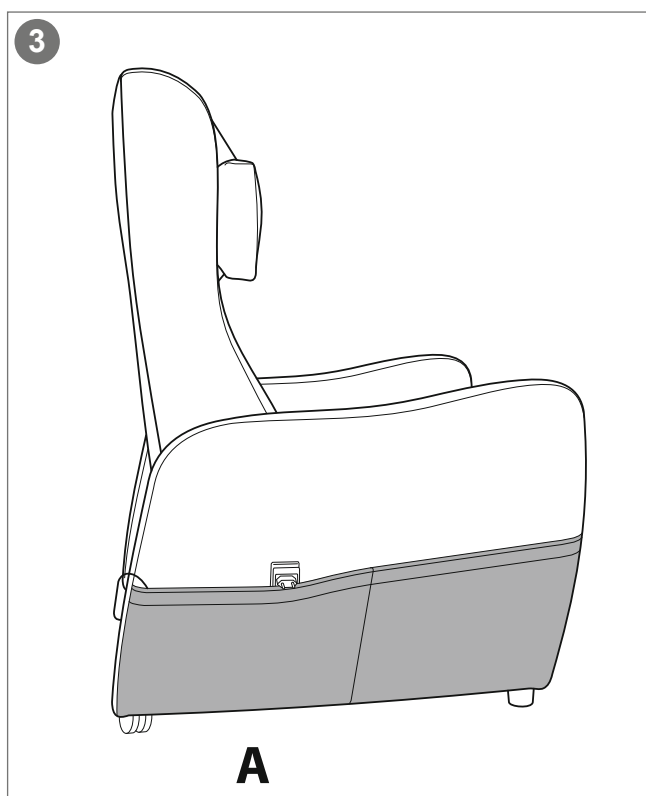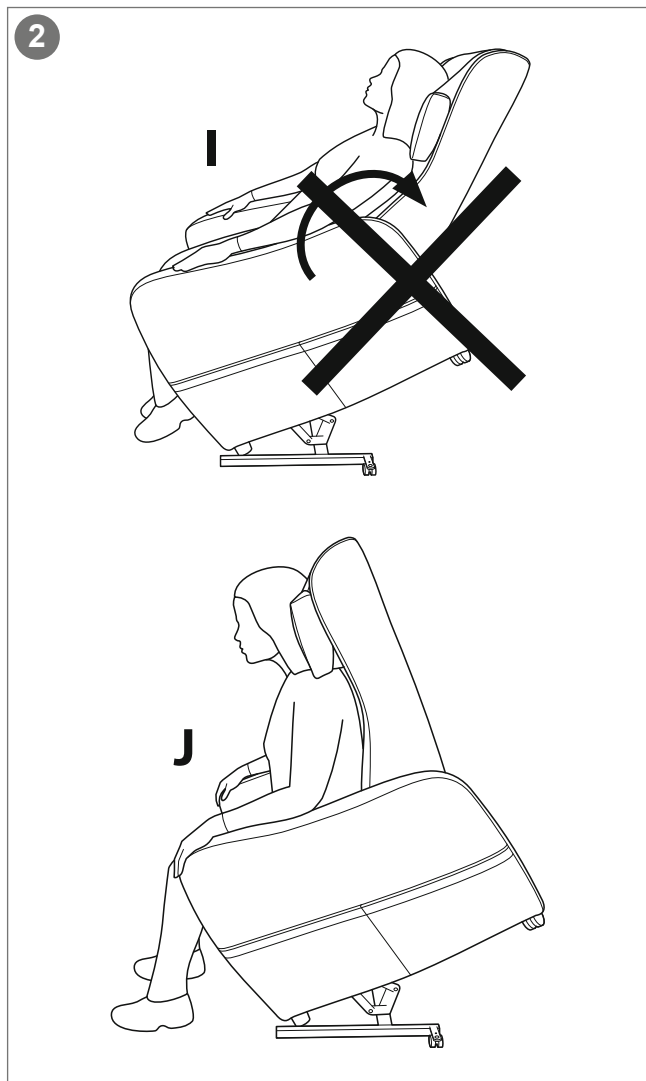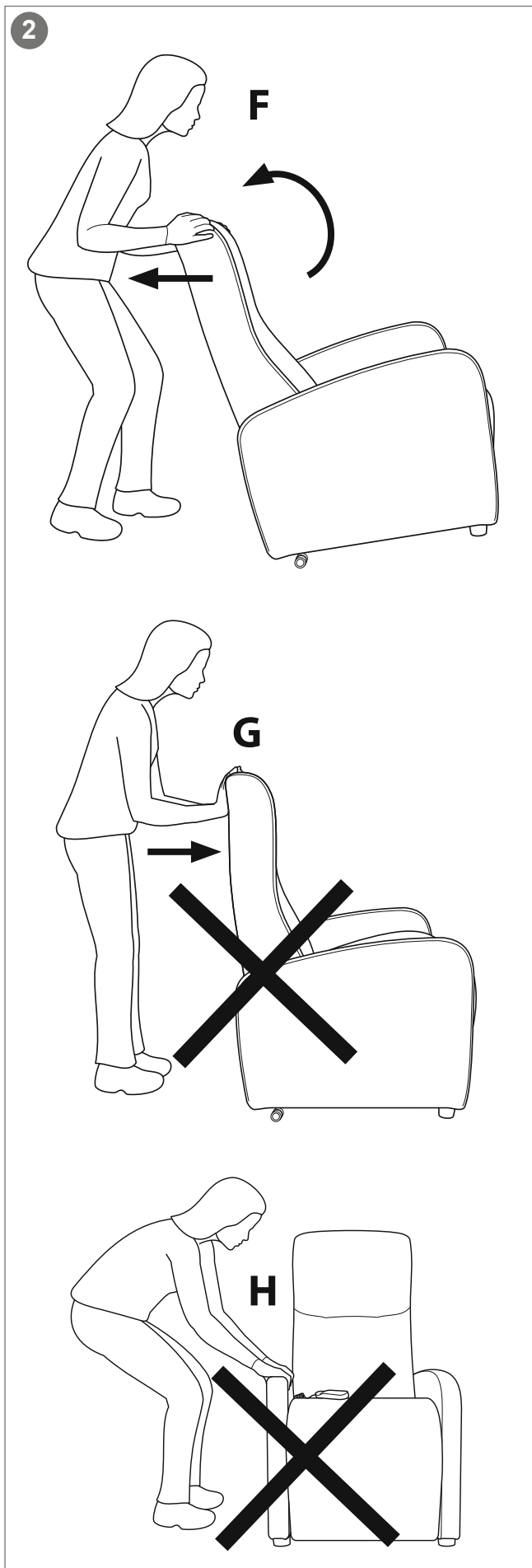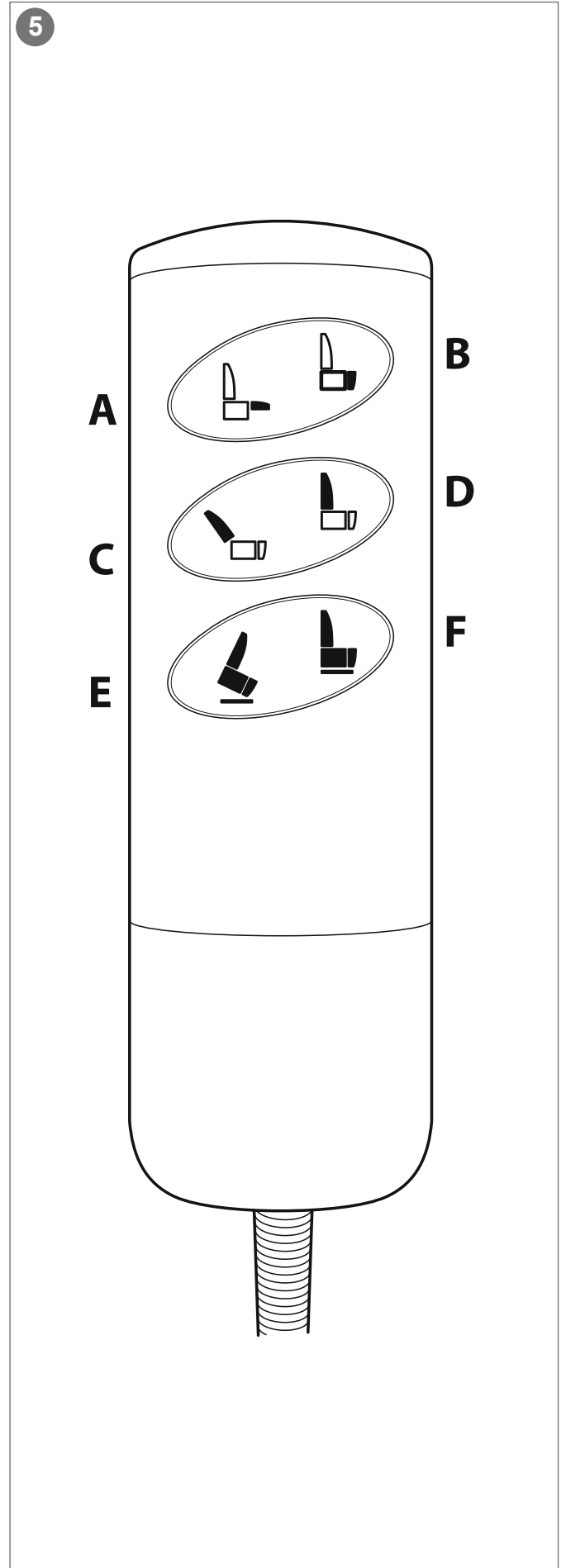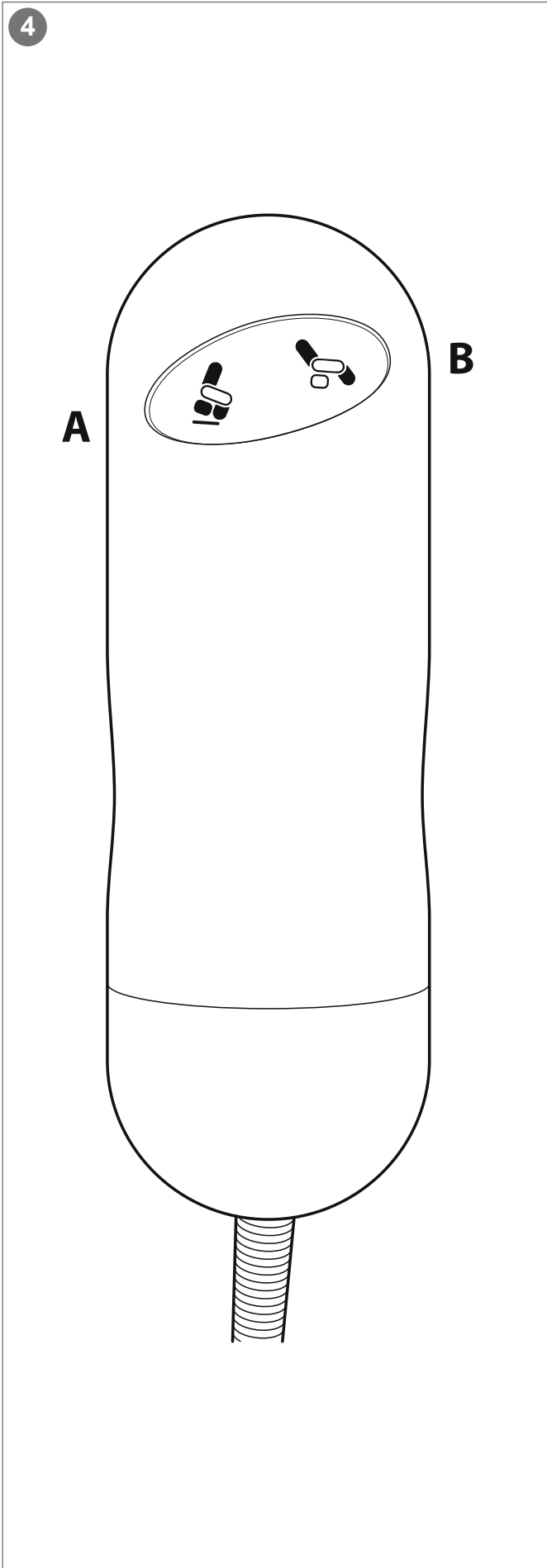Congratulations with your choice of a new TOPRO rise and recline chair

Designed with your comfort in mind, our rise and recline chairs offer exceptional support for your back and legs with their ergonomically designed seats, foot and backrests. TOPRO rise and recline chairs are available in a range of modern colours and beautiful upholstery, with features designed to enhance your comfort and safety.

On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. If you find it challenging to read this user manual, an electronic version is available on topromobility.com.

Intended use

TOPRO Comfort Basic and TOPRO Comfort Plus are intended for able-bodied adults, and the chairs should not be equated with wheelchairs, medical chairs or beds for disabled people. The chairs are intended for indoor use and can only be used by one person at a time within the weight limit of 120 kg. The user must be able to operate the product without external assistance. The warranty is void if the product is damaged by users who do not comply with these restrictions.

0 The grey numbered circles in this user manual refers to the numbered information graphic and illustrations inside the cover

1 Get to know your TOPRO product

- A** Back rest
- B** Seat
- C** Remote control
- D** Footrest
- E** Armrests
- F** Head rest (TOPRO Comfort Plus)
- G** Side pockets (TOPRO Comfort Plus)

2 Safety 

Never enter or exit the chair without first returning the footrest to its original position (A). The footrest is designed only for its intended use (B) and should never be used as a seat or for similar purposes (C), as the hardware of the product may be damaged.

The chair should be positioned in such a way that no obstacles can impede the operation of the lifting and reclining mechanism (D). Under no circumstances should the backrest or footrest be placed against a wall (E), due to the risk of significant damage that is not covered by the warranty.

The chair must be placed on a level floor.

When moving the chair, tilt and pull it backwards while holding the backrest (F). Do not push the chair forward (G).

Do not pull the chair sideways by its armrests (H).

Always take the precaution by disconnecting the power supply to the chair when cleaning or moving it, or when children are present unsupervised.

The chair is not a toy and should not be used by children. Be sure to use the chair with caution in the presence of children or pets.

Never recline the backrest (I) while the chair is in a raised position (J) due to the risk of injuries.

Do not pull the remote-control cord more than necessary. Do not sit on the remote control or let it fall to the ground. Do not allow children to use the remote control.

Carefully ensure that the product is properly installed and connected to the power supply before use.

Do not modify the product, as this may compromise your safety and void the warranty.

Check that the connectors under the chair are securely in place and not disconnected due to household cleaning, such as with a broom or vacuum cleaner.

Keep the chair away from light or heat sources.

3 Installation

Wired Power Supply

To activate the motors, plug it into the power outlet. Once connected to the socket, you can operate the remote control (see section 4 for TOPRO Comfort Basic and section 5 for TOPRO Comfort Plus).

When not in use the power supply can be easily placed in the side pocket (A).

Wireless Power Supply (TOPRO Comfort Plus accessory)

This option includes a battery which can be placed underneath the chair or in the side pocket.

 Read and carefully follow this information before using the wireless power supply option.

Initial Setup

Before first use, charge the battery for at least 24 hours. To do this, connect the charger to the battery correctly. After connecting the charger to the battery,

plug it into a power socket. The charger's LED light should turn green.

Battery Charging

When the battery is fully charged, the LED indicator on the battery will turn green. The average charging time can take approximately 3–6 hours.

To prevent premature battery wear and extend battery life, disconnect the charger when charging is complete.

The battery reaches full capacity after 5–10 charge and discharge cycles.

It may be necessary to perform several charge and discharge cycles after a long period of disuse to restore battery performance.

Safety – Battery and Battery Charger!

Due to chemical reactions, the battery heats up during charging. If it becomes very hot, the protection circuit is faulty. In that case, remove the charger and contact customer service.

While in operation, the battery can be stored in the chair's side pocket (A), but not during charging due to the high temperature of the battery.

Only use the charger sold with the battery.

Never use the charger in damp environments.

Charge the battery only at temperatures between 0°C to + 45°C.

Store the battery out of reach of children.

Do not open or damage the battery.

Do not expose the battery to a heat source or flames.

Do not immerse the battery in water.

Damage resulting from improper battery handling will not be covered by the warranty. Rechargeable batteries have a limited lifespan characterized by reduced charging capacity over time. Therefore, it is necessary to replace the battery approximately every two years.

4 Adjustments TOPRO Comfort Basic

- A Rise the backrest + Stand up support
- B Recline the backrest + Sit down support

5 Adjustments TOPRO Comfort Plus

- A Rise the footrest
- B Lower the footrest + Rise support
- C Recline the backrest
- D Rise the backrest
- E Stand up support
- F Sit down support + Rise the footrest

Troubleshooting

Problem	Probable Cause	Solution
The chair is not working	Battery discharged	Charge the battery
	No voltage	Check that the battery is connected to the motor(s) and that the electrical outlet is working
	Defective chair	Contact customer service
The chair suddenly becomes unresponsive, no movement takes place	The heat protection has been triggered	Leave the chair for 20 to 30 minutes and try again
	Battery discharged	Charge the battery
	A cable has been disconnected	Check that the battery is connected to the motor(s) and that the remote-control cord is connected
	The battery fuse may be out of order	Contact customer service

Please contact your supplier for further assistance if everything above has been checked and the chair still does not function properly.

Storage

For long-term storage, measures must be taken to protect and preserve the chair:

- Protect the chair from sunlight to counteract the natural process of colour fading by covering the chair.
- Store the chair in dry and well-ventilated rooms.

Cleaning

Protect the chair from all sources of direct light (sun, moon, halogen lamps, etc.) to avoid the risk of discolouration.

Use a vacuum cleaner and regularly wipe the chair without excessive friction to avoid damaging the upholstery. The vacuum cleaner will remove dry dust but not greasy dust that may adhere to the upholstery. Therefore, it is recommended to clean the chair every six months. Clean the parts in direct contact with the scalp and hands with water and a neutral soap. Avoid household detergents or products containing solvents or oils that can damage the upholsteries. Always test on a hidden part of the chair.

Oily materials	Use a dry cloth to absorb the oil and remove the stain. If this is not sufficient, apply a small amount of dish soap and gently rub with a damp cloth.
Coffee, tea and beer	Clean with hot water. If the stain persists, dab it with a soapy solution.
Chewing-gum	Place an ice cube wrapped in cloth to cool the gum. Then scrape it off very gently.
Ballpoint pen ink	Dampen a cotton swab with a little vinegar and run it over the stain.
Food stain and blood	Carefully remove the superficial part and moisten the stain with some water. Then dab with a cloth soaked in soapy water.

Tip: For tough stains, consult a stain removal specialist.

Inspection / Maintenance

The mechanism must be dusted regularly, and fasteners be checked and tightened if necessary (every year). During use, the

mechanism may play slightly, and minor sounds or creaks are unavoidable during operation. It is recommended to apply a small amount of lubrication to the joints

in such cases. Some coatings, especially faux leather, may cause squeaking during operation, and talcum powder is recommended to mitigate these effects.

Material

	TOPRO Comfort Basic	TOPRO Comfort Plus
Seat	Beech frame with metal springs High resilience foam 34 kg/m ³	Beech frame with metal springs High resilience foam 34 kg/m ³ Memory foam seat mattress 52 kg/m ³ , thickness 2.5 cm
Back rest	Plywood structure Elastic strap High resilience foam 26 kg/m ³	Plywood structure Elastic strap High resilience foam 26 kg/m ³ Memory foam seat mattress 52 kg/m ³ , thickness 2.5 cm
Footrest	Panel structure High resilience foam 34 kg/m ³	Panel structure High resilience foam 34 kg/m ³ Memory foam seat mattress 52 kg/m ³
Armrest	Plywood structure Polyether foam 20 kg/m ³	Plywood structure Polyether foam 20 kg/m ³
Head rest	Viscose fibers (Head rest only offered as accessory).	Viscose fibers

Tip: To reduce the firmness of the head rest, open the zip on the back and remove some viscose fibres.

Warranty

TOPRO Comfort Basic and TOPRO Comfort Plus is guaranteed free for faults and defects for 2 years. For repairs during the warranty period, please contact your local mobility shop or dealer. Our warranty applies in connection with the intended use according to this user manual.

Exclusions from Warranty

Unintentional damages caused during delivery or installation of the product by the distributor or user.

Damages caused by misuse of the chair.

Changes in appearance caused by normal use and wear of the product.

Products in abnormally dirty condition and/or with unintentional or deliberate damage.

Damages resulting from prohibited use: sitting on the footrest or armrests, entering or exiting the chair without first folding the footrest, moving the product by pulling sideways on the armrest or the backrest, obstructing the free movement

of the elements in the backrest or footrest, especially against a wall or furniture.

Presence of a mark on any part of the chair indicating it was placed against a wall, especially on the backrest, will invalidate the product warranty.

For the electrical part, damages caused by power surges or lightning, poor electrical installation, cutting or tearing of electrical wires, breaking connection plugs, unintentional disconnection of motors or remote controls.

Accessories

	TOPRO Item No.
Head rest, black faux leather (TOPRO Comfort Basic)	8453 HR
Rechargeable battery (TOPRO Comfort Plus)	8454 BAT

Accessories can be ordered separately to customize the chair to individual needs. Contact your mobility shop, dealer or TOPRO for an updated overview of available accessories or visit our homepage [topromobility.com](https://www.topromobility.com).

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TOPRO Mobility AB
strack AG